



Safety Precautions

Read all safety instructions before operating. Use this device ONLY for the purpose for which it is intended. Read Device Owner's Manual before using.

Contra-indications

- Do not use on persons with a demand-type cardiac pacemaker or other implanted electronic device.
- Do not use on a stroke patient until all possible blood clots have been resolved.

Warnings

- Do not rub over the carotid artery.
- Do not rub over areas such as phlebitis, thrombophlebitis, varicose veins, or over the vessels that may contain a blood clot.
- Do not use device on the eyeball or over closed eyelid.
- Do not use on a person during pregnancy.
- Stimulation should not be applied over the throat. Spasm of the laryngeal and pharyngeal muscles may occur, and the contractions may be strong enough to close the airway or cause difficulty in breathing.

Precautions

- Proper hydration is essential for optimal effect.
- Ensure skin is clean and free from lotions or moisture.
- Do not use while operating machinery or driving vehicles.
- Do not use the device on persons who may be under the influence of alcohol or other drugs.
- Use caution for persons with suspected or diagnosed heart problems.
- Use caution when there is a tendency to hemorrhage following acute trauma or fracture.
- Use device only with onboard electrodes or with leads and electrodes recommended for use by the manufacturer.
- Do not use in or near water.
- Do not operate if device has been dropped or damaged.
- Do not operate if device does not function properly.

Adverse Reactions

- Skin irritation and electrode burns are potential adverse reactions if the power is turned up too high.